



LOWER PRE-MOLAR CASE STUDY




1. Lower right second premolar site at 3 weeks post extraction.

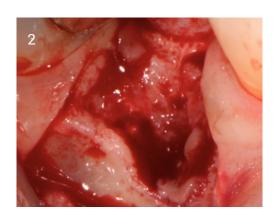

2. Site specific flap retaining the papillae, showing bone loss due to split root.

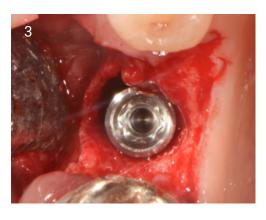

3. Implant placed in best position ready for grafting.

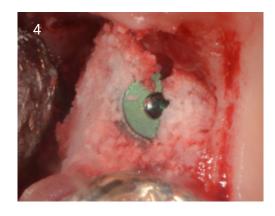

4. Buccal graft with EthOss® to regenerate bone.



5. Good soft tissue healing at 10 weeks and restoration of the ridge.



6. Flap raised to show newly regenerated bone at 10 weeks.


A biphasic matrix for true bone regeneration. Built in membrane function, high graft stability. **Ethoss Regeneration Ltd**8 Ryefield Court, Silsden, BD20 0DL, UK
www.ethoss.dental • info@ethoss.dental



7. Site of core sample.



9. Osstell meter reading at loading.

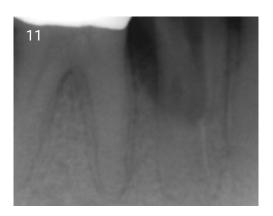

11. Radiograph showing split root and resultant bone loss.


LOWER PRE-MOLAR CASE STUDY



8. Core sample taken above the implant to gain access, sent for Histologic analysis.



10. Case at 6 months.

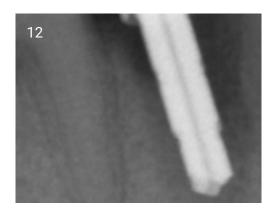

12. Radiograph of osteotomy creation.


A biphasic matrix for true bone regeneration. Built in membrane function, high graft stability. **Ethoss Regeneration Ltd**8 Ryefield Court, Silsden, BD20 0DL, UK
www.ethoss.dental • info@ethoss.dental



LOWER PRE-MOLAR CASE STUDY




13. Radiograph of placed implant with graft material.

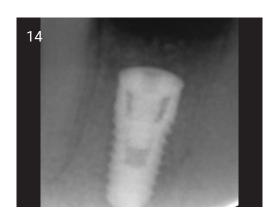

14. Radiograph at 10 weeks post placement showing raised bone level.



15. Radiograph at 6 months.

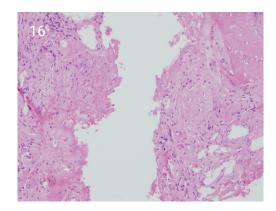

16. Histology by Prof. David Mangham, University of Birmingham, UK, showing approximately 50% (overall) induced woven bone.

Case Study Originally
Printed in European
Journal for Dental
Implantologists, 4/2014
pp 58 - 62.

Case by Dr Peter Fairbairn, London, UK