

Case Background

Female
54 years old
Periodontitis in tooth 43
has caused bone loss
Loosening of teeth: 3mm

Pocket depth: 9mm

Case performed by Dr Manuel Bras Da Silva, Lünen, Germany

PERIODONTITIS CASE STUDY



1. Pre-op view.




2. Severe bony defect.



3. Grafted with EthOss®, allowed to set for 3 minutes.



4. Sutured closed after graft material has set.

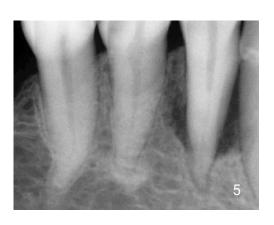

5. X-ray showing defect, before grafting.

A biphasic matrix for true bone regeneration. Built in membrane function, high graft stability. **Ethoss Regeneration Ltd**8 Ryefield Court, Silsden, BD20 0DL, UK
www.ethoss.dental • info@ethoss.dental



6

6. X-ray immediately after grafting with EthOss®.


Results after 12 months

Loosening of the teeth: 0-1mm

Pocket depth: 4mm No complications in healing

Patient reports no pain

PERIODONTITIS CASE STUDY

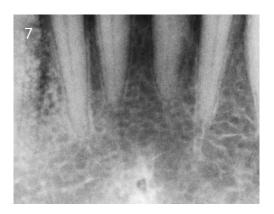

7. X-ray after 1 year showing new host bone.